

C.O.C. FARMACEUTICI SRL					
ARTICOLO Foglietto Illustrativo		PRODOTTO Schema f.to 130 x 180 mm			
DIMENSIONI TOTALI 130 x 180 mm		PIEGATO	PASSO	CODICE 000000	SCALA 1.1
CARTA 000000000		GRAMMATURA gr. 0	TIPO PRODOTTO		F.TO BOBINA Ø 00 mm Ø 000 mm

Edizione	Data allestimento
0	09-12-2015

L'impianto stampa non deve assolutamente essere modificato o danneggiato

VIZcellose 1.0% Carmellose Eye Drops

READ CAREFULLY BEFORE YOU START TO USE THE PRODUCT

VIZCellose 1.0% is a sterile, ophthalmic extra strong moisturising and protective solution, with physiological pH, based upon carmellose.

Thanks to the OSD closure system- able to preserve the microbiological, chemical and physical characteristics of the solution - VIZCellose 1.0% does not contain preservative. The absence of preservatives allows the use of the solution by contact lens wearers without the need of removing them.

Carmellose (CMC) is a polysaccharide known as mucilage. It increases the residence time of tears as well as increase the viscosity of tear film. Thanks to its proprieties carmellose protects the eyes against irritation and moisture the ocular surface, relieving discomfort sensation by creating a protective, transparent and viscoelastic shield. It helps the restore of the natural tear film for a lasting relief.

VIZcellose 1.0% relieves symptoms of dry eyes such as burning, itching, and foreign body sensation. VIZcellose 1.0% also acts as lubricant, providing relief of dry eye conditions, including those associated with the use of PC, infrequent blinking, certain medical treatments, atmospheric pollution, dry atmospheres (e.g. air conditioning, wind and sun) and ophthalmic stress due to the use of eye lenses.

VIZcellose 1.0% with 1.0 % of carmellose can be suggested in case of a severe dry eye syndrome or in case of necessity more immediate protective action and lasting relief.

INSTRUCTIONS FOR USE

- tear off the tamper-evident seal before the first administration;
- remove the protective cap and open the bottle;
- instill 1-2 drops in each eye slightly pressing the bottle;
- after use, close the bottle: the content should be re-used within the next 3 months from the first opening;
- it is suitable for frequent use. Unless your doctor has told you differently, you can use the product even several times a day (3 or 4 time or more often if required) to provide sufficient hydration. As dry eye is a condition which varies from person to person, the drops should be used as often as you think necessary, to relieve your dry eye.

The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions.

COMPOSITION

1.0% carmellose sodium (carboxy methyl cellulose), boric acid, sodium tetraborate decahydrate, sodium chloride, W.F.I. Preservative free.

WARNINGS AND CAUTIONS

- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Never use the product if the container results, at the time of first opening, damaged or not perfectly sealed.
- During the application, do not touch your eyes and any other surface with the dropper of the container.
- Contraindicated in patients with a history of hypersensitivity to any of the components.
- In case of problems during treatment interrupt the use and consult a doctor.
- Do not use after the expiry date indicated on the package and label.
- Dispose of properly the container after use.
- Keep away from children.

COMMON SIDE EFFECTS

- The product can cause a short term blurring of vision after use. Do not drive or use machines until your sight is clear.
- Like all eye drops, there is a slight possibility that the product can occasionally cause mild eye irritation or stinging or itching, but this usually soon passes and it can occur in a very small number of people.

If any of these effects persist or worsen, contact your doctor promptly.

PACKAGE

10 ml Multi dose bottle, preservative free.

STORAGE CONDITIONS

Do not store above 30°C.

After the first opening the content must be used within 3 months.

VALIDITY

With un-open pack, not exceed the expiry date stated on the box and on the bottle label.



C.O.C. Farmaceutici S.r.l., Via Modena, 15,
40019 – Sant' Agata Bolognese (Bo), Italy.



MEDICAL DEVICE



10ml

Last revision 01.01 10/2015

VISTO SI STAMPI

C.O.C. FARMACEUTICI SRL